orthocaps[®]

The TwinAligner[®] System

Welcome to orthocaps The TwinAligner® System

Through today's advertising driven world patients are becoming aware of the latest treatments available to them and are demanding more from their dentists. It is important that the treatments not only work, but are aesthetically acceptable and allow the patients to continue with a normal life. Patients are regularly asking for specific treatments; „Can I have invisible braces?’ They know that they no longer have to wear fixed braces which are both uncomfortable and unsightly, to have their teeth straighten.

orthocaps is a system that uses low forces and softer plastics to move teeth efficiently. It is a treatment system that is both effective and invisible. orthocaps can be used to treat most of all malocclusions, specially on adults.

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What is orthocaps?

orthocaps is for those patients who want a gentle, fast and less conspicuous way of straightening teeth. orthocaps provides a series of soft and hard CAPS, that after a period of time will straighten malocclusions and other tooth discrepancies.

The system was developed by orthodontist Dr. Wajeeh Khan. Dr. Khan and his team are experienced practising orthodontists who act as consultants throughout the entire treatment. Having worked with alternative systems, Dr. Khan was not entirely happy with the results, as he considers the consistent use of aligners made up of hard plastics to be too severe.

The use of hard aligners results in heavy forces which are counter-productive and sometimes harmful for teeth and surrounding tissues.

orthocaps is a system that uses low forces and softer plastics to move teeth efficiently. The softoCAPS as the name suggests, are made of a softer thermoplastic material and are designed for night time wear, giving light and gentle forces, which are better for orthodontic treatment.

Our philosophy is to achieve the best possible results by using the right appliance at the right time for each individual patient.

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What's different about orthocaps?

Each and every treatment is diagnosed and planned by our consultants, who are experienced and practising orthodontists, together with the clinician. For this purpose orthocaps uses a computer based diagnostic technology and an internet portal for communication with the clinician.

The internet portal specially designed for orthocaps is not only used for communicating and transmitting patient records to orthocaps but also as an effective interactive communication tool between the clinician and the consultant. By using this telemedicine facility it is possible to receive treatment guidance from experienced orthodontists not only for diagnosis and treatment planning but also during the entire treatment process.

By using two different types of thermoplastic materials to make the Caps (softoCAPS, hardCAPS), orthocaps is achieving markedly improved final treatment results as compared to other systems. orthocaps is the only system which uses two different types of appliances (softoCAPS, hardCAPS) for each treatment stage. This method helps to keep the orthodontic forces low and therefore results not only in a gentler and comfortable treatment for the patient, but also ensures more accurate tooth movements.

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What are the advantages?

- High quality tooth regulation
- Low forces that help to achieve better clinical results
- Treatment guidance and treatment monitoring by a team of experts throughout the entire treatment
- Latest technology - telemedicine techniques
- Interactive 3D digital set-up for treatment approval (iSetup®)
- An appliance that is comfortable for the patient
- Cost-effective

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What are the advantages for my patients?

orthocaps is a truly beneficial treatment option for your patients. The CAPS are almost invisible and are perfect for adult patients or for those who don't want to reveal that they are wearing an appliance.

Our cutting edge technology allows for shorter treatment times and you can give your patients a series of CAPS to progress with themselves and therefore reduce the number of appointments needed. It is very convenient for those patients who work away from home or have difficulties meeting regular appointments.

We use soft thermoplastic materials that are 100% bio-compatible for our appliances and each appliance is customised to your patient's requirements, providing an exact fit. This means treatment is painless and the appliance fits comfortably.

Overall, your patient is looking for a successful aesthetic outcome and orthocaps enables you to give patients this from the start - aesthetically pleasing appliances with amazing final results!

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Treatment with orthocaps

Treatment with orthocaps will vary according to the severity of the case. On average there are 12 - 18 sets of CAPS per treatment with an average of 3 weeks wearing time per set of CAPS. orthocaps can be used for 80% to 90% of all orthodontic treatments. Skeletal cross bites, surgery cases and certain rotations are not suitable for treatment with orthocaps.

A range of treatment options are available with orthocaps:

- CAPS for anterior teeth (canine to canine) for one jaw, upper or lower, 6 teeth in all
- CAPS for anterior teeth (canine to canine) for both jaws, 12 teeth in all
- CAPS for all teeth (single jaw), upper or lower
- CAPS for all teeth, both jaws

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orthocaps in practice

The orthocaps portal can be used for treatment evaluation. For this purpose a set of extra- and intraoral photographs are transmitted to orthocaps. Within 24 hours a response is sent to the clinician saying if the patient can be successfully treated with the orthocaps system. This facility is free of charge for certified orthocaps practitioners.

After making the decision to use orthocaps, patients will need to have the following procedures:

- Silicon-A impressions of the patient's upper and lower jaw
- Silikon-A bite registration
- Photographs
- Panoramic X-ray

The impressions are shipped to Germany. After treatment planning a digital 3D model of the treatment goal is transmitted to your account for you and for your patient to view. After you have approved this treatment goal, orthocaps will go ahead with the manufacturing of the CAPS. You will receive the first set of CAPS for the first treatment phase which comprises of 8 treatment stages within 2 - 3 weeks.

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I'm interested in orthocaps - How do I proceed?

- Visit the orthocaps website
www.orthocaps.com
- Go to 'Information for Dentists'
- Login
- Sign in as a clinician
- Enter all required details
- Submit your application

At this point your screen will return to the original opening page where it will ask you for your user ID and password. You will need to wait for an email to confirm your ID and password before you enter the portal.

Now you are registered with orthocaps.

This registration process opens an orthocaps account for you.

Before you can start using the system you need to take part in a certification course.

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How do I become a certified orthocaps practitioner?

orthocaps runs certification courses in German and English on a regular basis.

The details of these courses can be found on the orthocaps website. For further information please contact our sales and marketing director Jocelyne Khan at +49 (0)2385 92190 or info@orthocaps.com

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Ortho Caps GmbH